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Produktbeschreibung

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EU Instructions for use

Product description

HydroTac is a foam dressing with a gas-permeable, waterproof and bacterial resistant outer layer made of polyurethane. The dressing face in contact with the wound, is impregnated with a hydrogel. The gel consists of a water containing polyurea / hybrid polyurethane-polymer, which contains propylene glycol.

In addition, HydroTac comfort/HydroTac sacral have an adhesive border, which is coated with polyacrylate adhesive.

HydroTac (comfort) is for the treatment of wounds with moderate to severe exudate.

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HydroTac concave: Remove the transparent protective foil following the numbering (1 & 2), place the dressing with the gel-impregnated side on the wound and press down the adhesive edges firmly. (Take care that no pressure is applied to the wound.) Remove the protective top layer following the numbering (3 & 4).

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HydroTac sacral: Remove the transparent backing foil number 1, place the dressing with the gel-impregnated side on the wound and press down the adhesive edges firmly. Then remove the remaining backing foils number 2 on both the sides and press down the adhesive edges. Remove the protective top layer following the numbering (3 & 4).

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General instructions
• Store between 5 and 40 °C.

- * Keep out of the reach of children.

* Any reference to HydroTac contained in the text, refers to all presentations, unless otherwise specified.

Date of revision of the text: 2018-03

Single use only

-  Do not re-sterilize

Do not use if the package is damaged

-  Keep dry

HydroTac can absorb wound exudate and release moisture even when applied to dry wounds. This provides a moist wound environment which has a stimulating effect on wound healing. The gel on the side of the wound-contact layer prevents the dressing from sticking to the wound. It can be removed almost painlessly.

HydroTac (comfort) is intended to be applied by professional users. Wound care professionals may delegate dressing changes to patients and/or their relatives after appropriate training.

HydroTac (comfort) is intended to be applied by professional users. Wound care professionals may delegate dressing changes to patients and/or their relatives after appropriate training.

HydroTac/HydroTac concave: Remove the transparent protective foil following the numbering (1 & 2), place the dressing with the gel-impregnated side on the wound. The self-adhesive properties make HydroTac/HydroTac concave easy to apply.

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