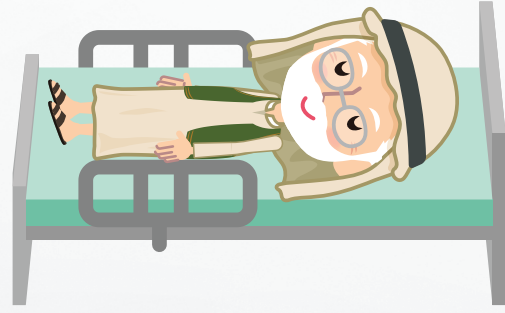
## Application Guide

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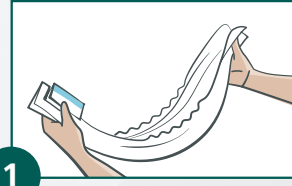
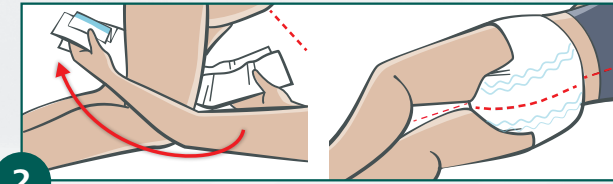
Gently fold it in length to activate the inner cuffs and create an anatomical 'cup'

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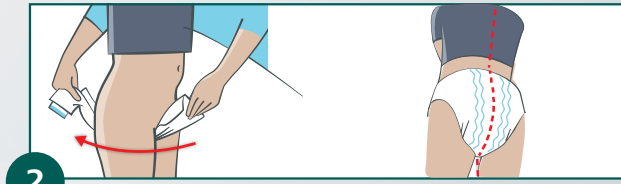
Turn the patient on the side and slide it from the front to the back

Center the pad by aligning the wetness indicator with the spine. Push the lower side panel under the hip and turn the patient onto their back.

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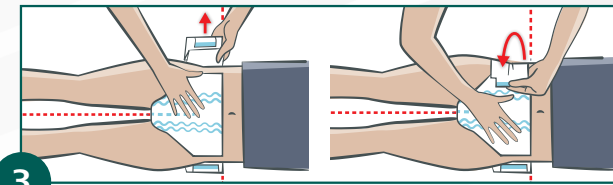
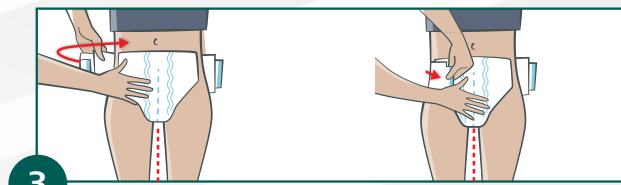
Stand beside the patient. Slide it from the front to the back. Center the pad by aligning the wetness indicator with the spine.

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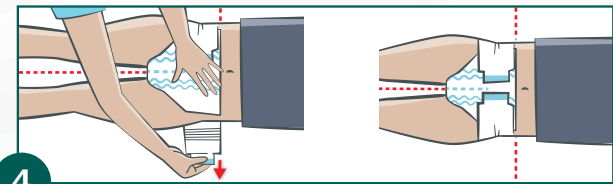
Fold out the 1st side panel. Wrap around the hip and stick it.

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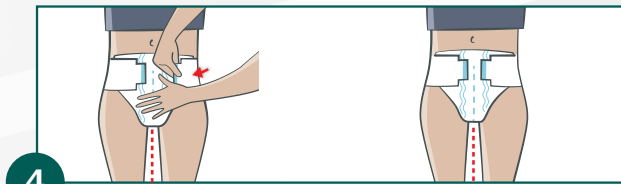
Fold out the 2nd side panel. Wrap around the hip and stick it.

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### Sizing Tips:

- Measure the wait and hips
- Use the larger size to choose correct product size
- If between sizes, try the smaller size first

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References  
(1)The reduction from 4 to 2 tapes (one for each side) combined with no side panels in the front, simplifies the application, by reducing the application steps from 8 to 4 compared to a traditional slip with 4 side panels and 4 tapes. Additionally, due to the elasticated side panels, there is less adjustment needed. (2)Laboratory report about ergonomic evaluation of MoliCare® variants A+B incontinence products. Research conducted between 5-8 October 2015 in Dijon, France. W. Schneider, H. Mutschler, R. Haller, C. le Mathe. Comparison with a slip having 4 side panels and 4 tapes. (3)Bliss et al., Absorbent Briefs Containing Curly Fiber Lower (Acidify) Skin pH Reducing Risk for Incontinence Associated Skin Damage in Older Nursing Home Residents; WOCN Conference Symposium 2016. (4)Serazhudinova, N. Isaeva, E. Kolesnikova, A. Farafonova, J. Eilers, A. Ebert, R. Kesselmeier, H. Smola: Skin health and incontinence – characterization of specialty cellulose fibers for diaper design with antimicrobial properties; 2016

PAUL HARTMANN Middle East  
Dubai Airport Freezone  
P.O. Box 54525 Dubai  
United Arab Emirates  
Phone: +971 4 2996996

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MCCEMNNKVT0109/20

# Get upgraded with MoliCare® Premium Elastic

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Synchronized with body movements

Adapts to all body movements and stays in place due to its Elastic-Fix System

Simplified application

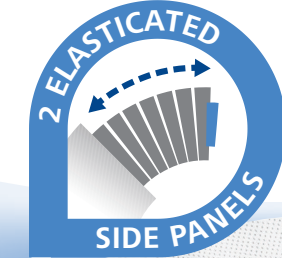
Application in only 4 steps instead of 8<sup>(1)</sup>

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2 RE-CLOSABLE TAPES

4 STEPS ONLY

Active Skin Protection due to the Curly Fibers, with the potential to prevent, reduce severity, and promote healing of Incontinence Associated Dermatitis<sup>(3)</sup>  
Inhibits the growth of pathogenic bacteria<sup>(4)</sup>

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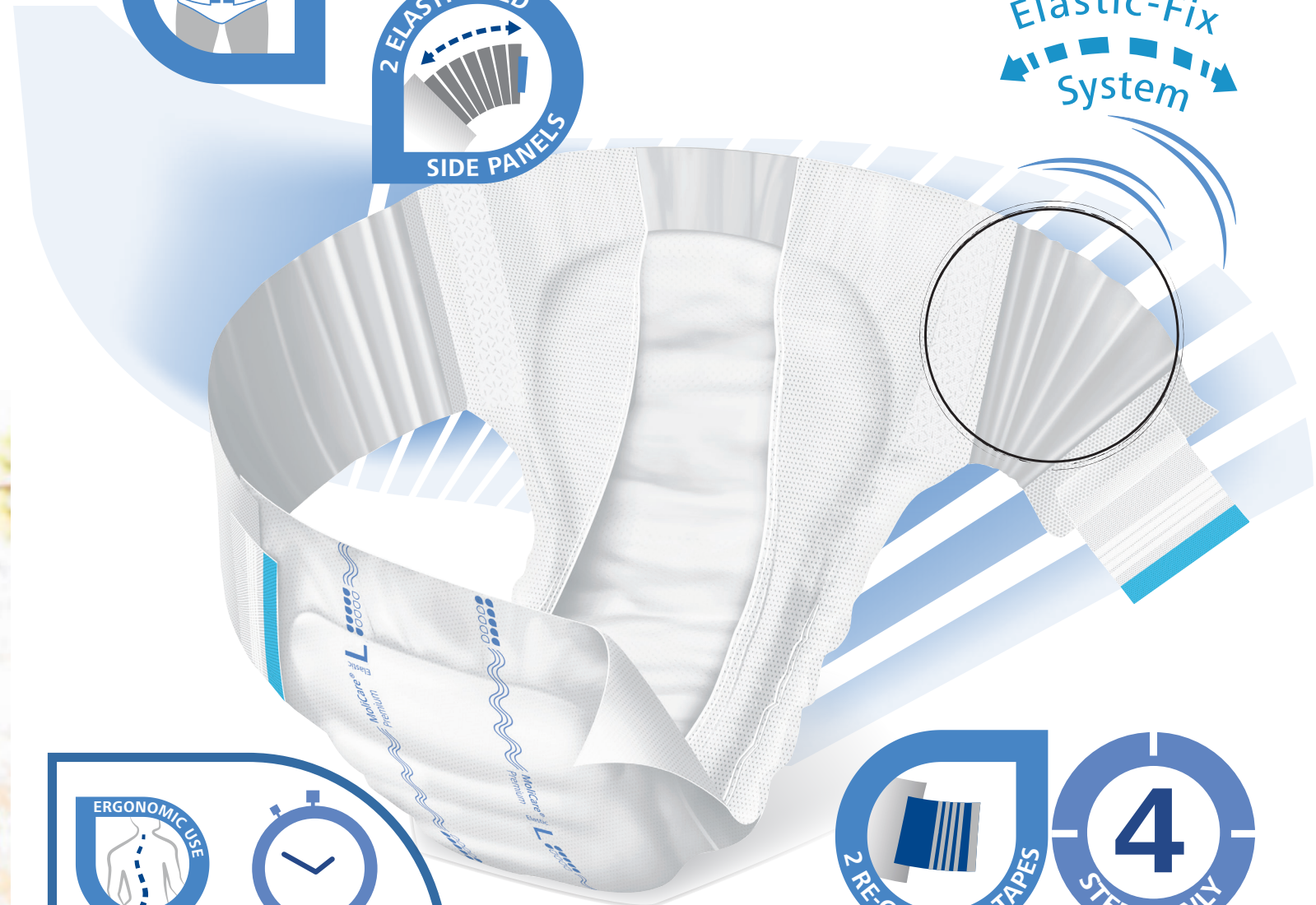
QUICK DRY SYSTEM

HYPOALLERGENIC  
DERMATOLOGICALLY TESTED

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Easy quick and mild cleansing for incontinent persons, especially heavy soiled skin with faeces.

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Quick and practical incontinence cleansing with Chamomile Recucita Extract that soothes irritated skin.

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"MoliCare®'s active skin protection concept Skintegrity helps to avoid incontinence associated dermatitis (IAD). Together, the skin care & absorbent products were designed to prevent skin irritations, one of the most frequent complications of incontinence."

Prof. Dr. Hans Smola, dermatologist, University of Cologne/Germany and head of Med Science, PAUL HARTMANN AG

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Intensive care for sensitive elderly skin